




3Shape Communicate

Setup Instructions

Step 1: 3Shape Communicate Account Setup


1. Go to 3Shape Communicate Web Portal: <https://portal.3shapecommunicate.com/login>. Click on **Create 3Shape Account**.

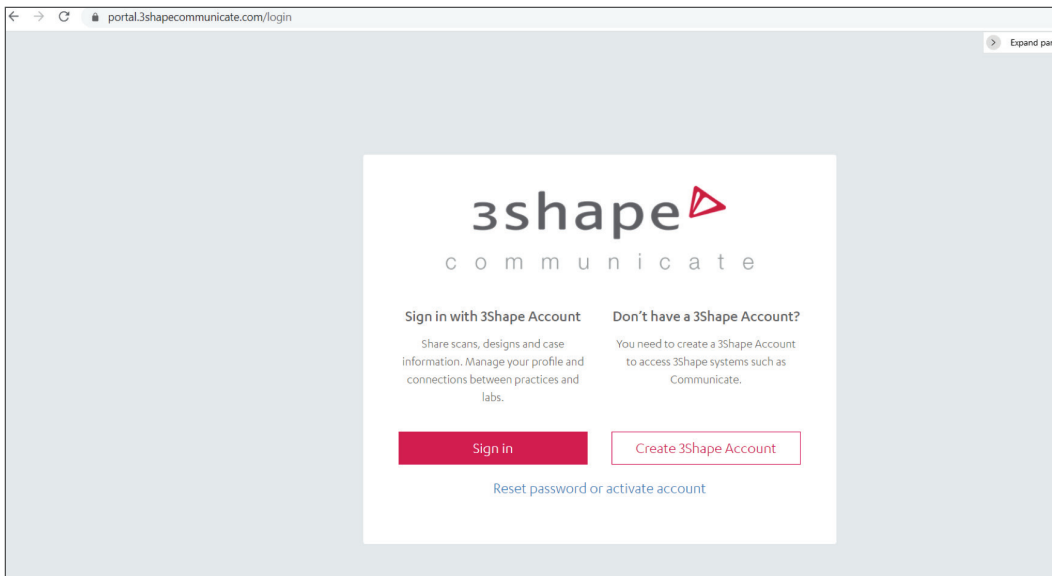

2. Enter your Details (First Name, Last Name, Email and Password) in the window below.

Step 1: 3Shape Communicate Account Setup (continued)

3. You will receive a verification email in the inbox of the email address you entered.

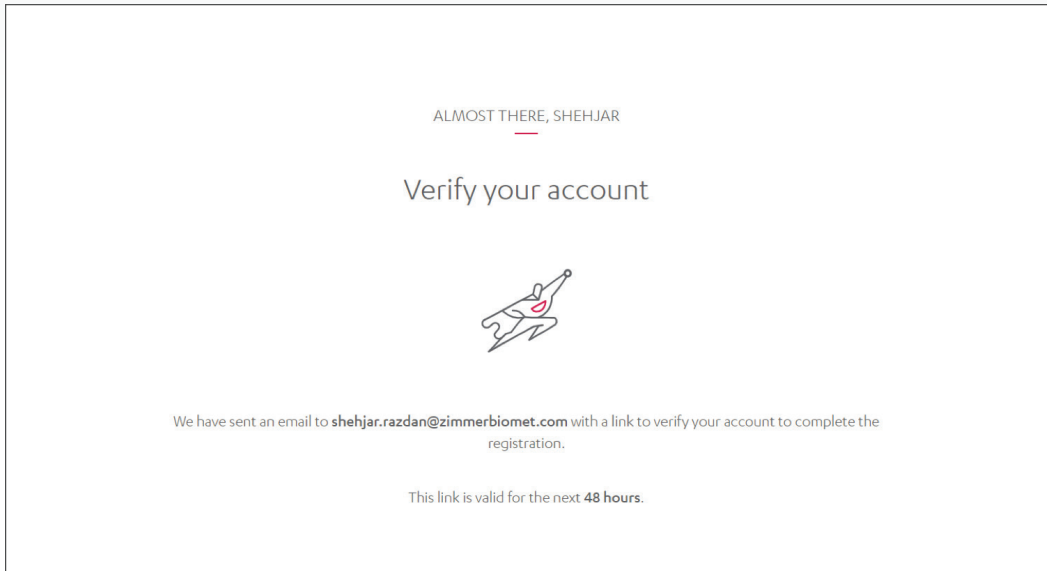

4. Once verified, please set up your Profile entering your Company as well as Contact information and all the required fields in the two windows below:

1 Profile — 2 Contact — 3 Completed

Set up your company account

Company name *
This field is required

Company type *
 Clinic
 Lab

Company email *

Company website

* Required fields

NEXT

Step 1: 3Shape Communicate Account Setup (continued)

The screenshot shows a three-step progress bar at the top: Step 1 (Profile) is checked, Step 2 (Contact) is active, and Step 3 (Completed) is not. Below the progress bar, the text "You are almost there!" is centered. The form contains the following fields: "Country*" with a dropdown menu showing "United States of America"; "State*" with a dropdown menu showing "Florida"; "Address*" (empty); "City*" and "Postal code*" (empty); and "Company phone number*" with a dropdown menu showing "+1". A red "REGISTER COMPANY" button is located at the bottom center.

5. Once completed you will receive a Company registration confirmation as shown below.

The screenshot shows the same three-step progress bar, but now Step 2 (Contact) is checked and Step 3 (Completed) is active. The text "Congratulations! Your company is registered!" is centered. Below the text is a circular icon of a building. Underneath the icon, the name "Shehjar Razdan" and the role "Administrator" are displayed, with a green checkmark next to "Administrator". A red "NEXT" button is located at the bottom center.

Step 1: 3Shape Communicate Account Setup (continued)

6. You can add your colleagues to your Company 3Shape account by adding their email addresses in the window below.

Invite your colleagues!

Enter emails of people you would like to add to your clinic or lab.

Member ▼

+ Add another

[SEND INVITATIONS](#)

[INVITE LATER →](#)

Step 2: Adding ZimVie Dental BellaTek Production as your Partner on 3Shape Communicate

1. Sign in to your 3Shape Communicate account using the credentials created earlier and agree to the standard 3Shape terms and conditions mentioned below.

3shape

c o m m u n i c a t e

Sign in with 3Shape Account

Share scans, designs and case information. Manage your profile and connections between practices and labs.

[Sign in](#)

Don't have a 3Shape Account?

You need to create a 3Shape Account to access 3Shape systems such as Communicate.

[Create 3Shape Account](#)

[Reset password or activate account](#)

Step 2: Adding ZimVie Dental BellaTek Production as your Partner on 3Shape Communicate (continued)

3Shape is requesting your permission

* Required fields

I agree to 3Shape's [Software-as-a-Service terms](#) *

I agree to the policy [Data Processing and Business Associate Agreement](#) *

3. Once you have accepted the terms and conditions, you will be lead to the following Case Dashboard.

State	Patient	OrderNo	Collaborator(s)	Units	Tooth	Design Type	Received On
No orders to display for America							

4. Click on the profile button on the right side of the screen, click on 3Shape Account.

My Account

SR


Shehjar Razdan

3Shape Account Sign out

[Privacy Policy](#)

Step 2: Adding ZimVie Dental BellaTek Production as your Partner on 3Shape Communicate (continued)


5. Ensure that the Profile information is correct.

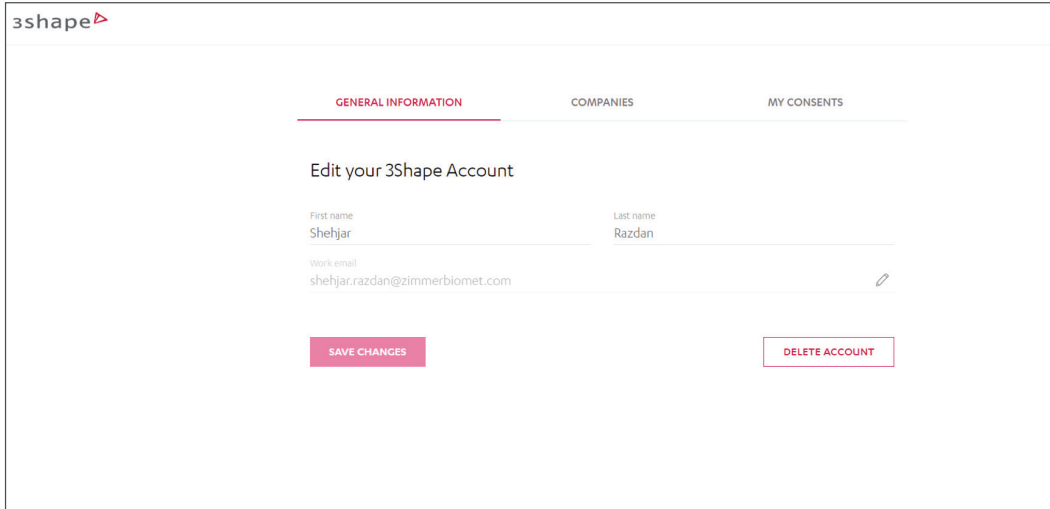

The screenshot shows the 3Shape account profile page. At the top left is the 3shape logo. Below it are three tabs: GENERAL INFORMATION (highlighted in red), COMPANIES, and MY CONSENTS. The main heading is "Edit your 3Shape Account". There are three input fields: "First name" with the value "Shehjar", "Last name" with the value "Razdan", and "Work email" with the value "shehjar.razdan@zimmerbiomet.com". At the bottom, there are two buttons: "SAVE CHANGES" and "DELETE ACCOUNT".


6. Click on the Connections Tab on the 3Shape Communicate Dashboard. In case ZimVie BellaTek Production Center sends you a 3Shape communicate connection request, you will see the request as shown in the image 6.1 below.

Please click accept to add us as a connection. Once added, it will display as an Active connection as shown in image 6.2.

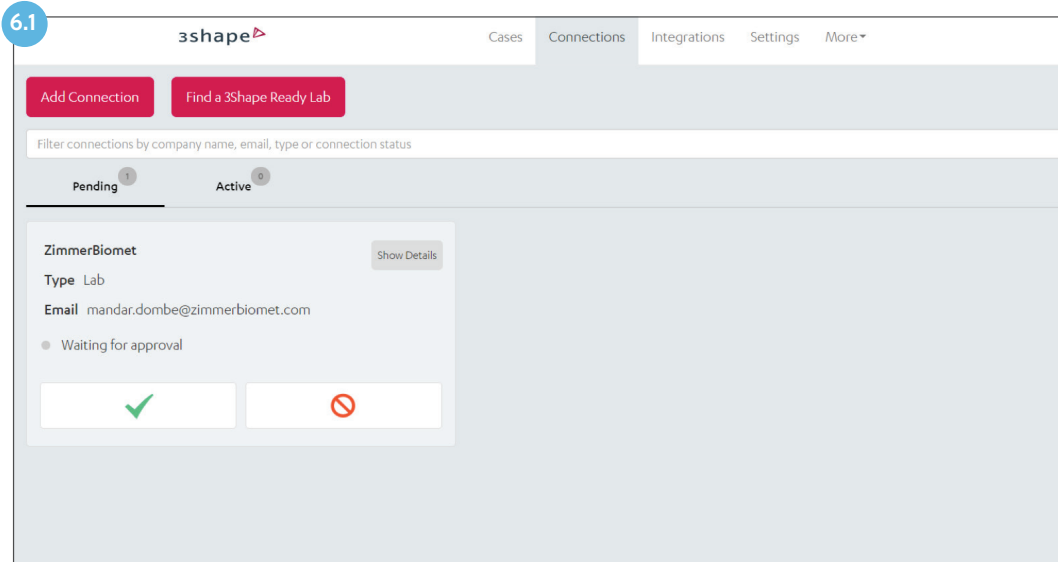


The screenshot shows the 3Shape Connections dashboard. At the top left is the 3shape logo. Below it are navigation tabs: Cases, Connections (highlighted), Integrations, Settings, and More. There are two buttons: "Add Connection" and "Find a 3Shape Ready Lab". Below these is a search bar: "Filter connections by company name, email, type or connection status". There are two tabs: "Pending" (with a count of 1) and "Active" (with a count of 0). The "Pending" tab is selected, showing a connection request from "ZimmerBiomet". The request details include: "Type: Lab", "Email: mandar.dombe@zimmerbiomet.com", and a status of "Waiting for approval". At the bottom of the request card, there are two buttons: a green checkmark button (Accept) and a red prohibition sign button (Decline).

Step 2: Adding ZimVie Dental BellaTek Production as your Partner on 3Shape Communicate (continued)

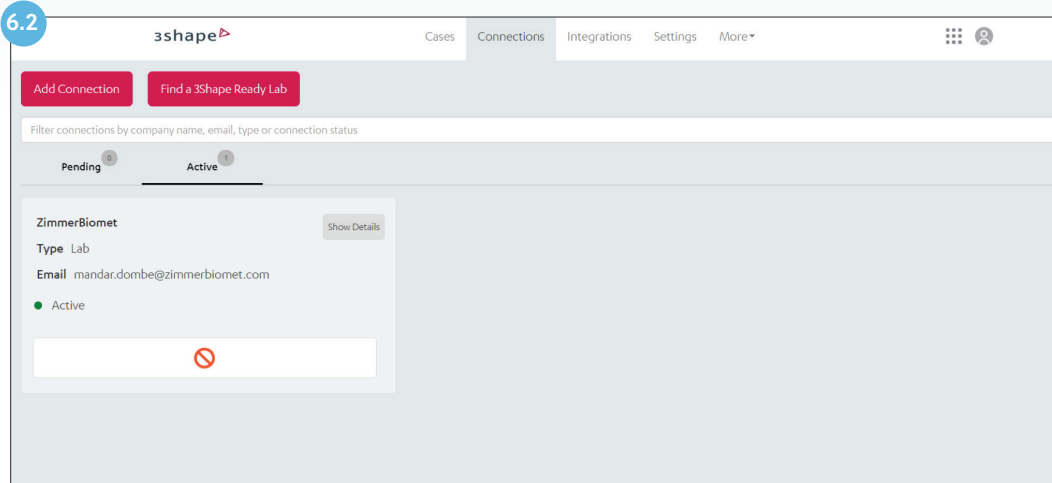


7. To send a connection request to ZimVie Dentak BellaTek Production Center, go to Connections Tab → Select Add Connection.

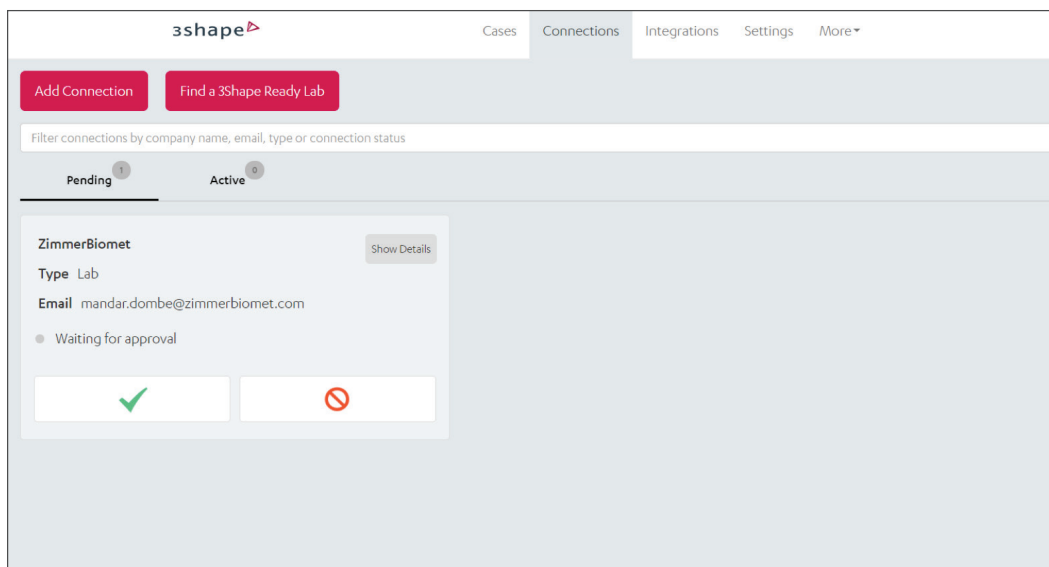


Step 2: Adding ZimVie Dental BellaTek Production as your Partner on 3Shape Communicate (continued)

- Once add Connection is selected, add the following email address: architectpsrdept@zimviocom to add ZimVie as a manufacturer.

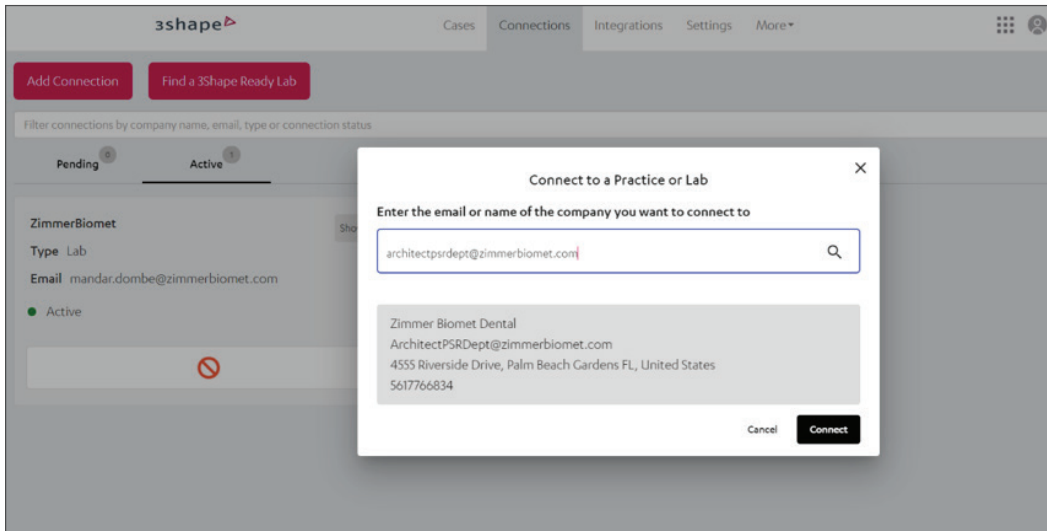


Step 3: Configure 3Shape Dental Manager for Communicate

- Launch 3Shape Dental Manager → Control panel → Tools → 3 Shape Communicate.

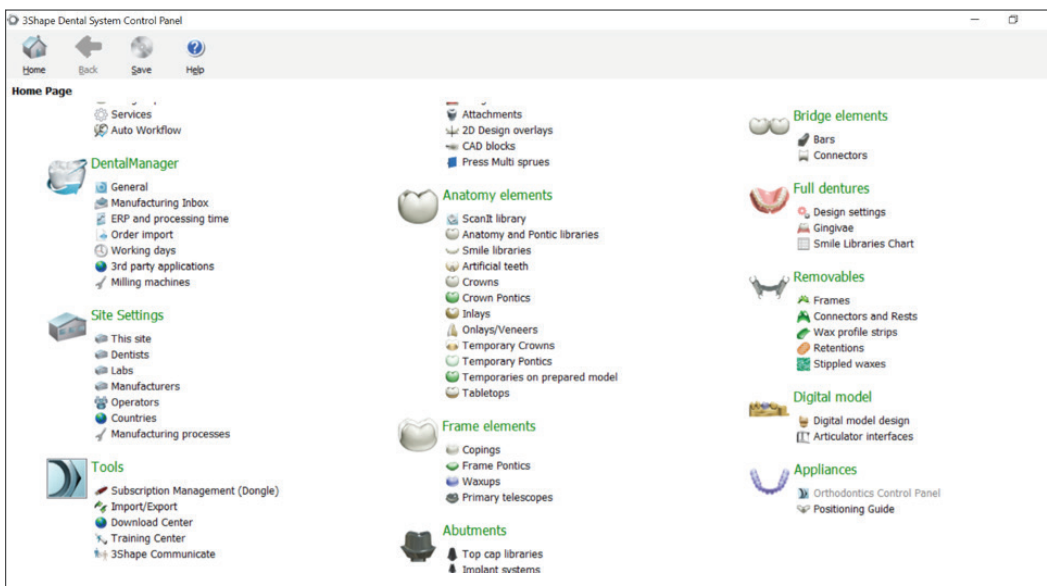


Step 3: Configure 3Shape Dental Manager for Communicate (continued)

2. 3Shape Communicate integration → Configure.

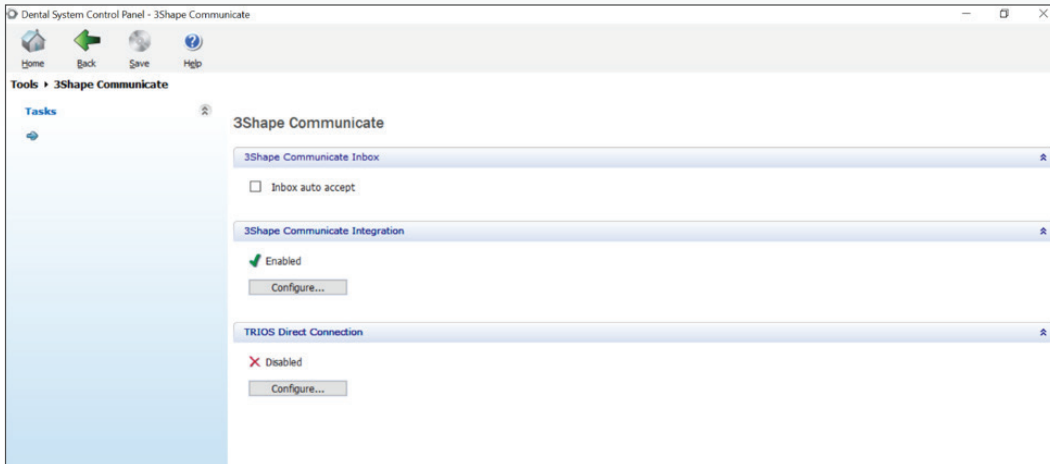

3. Click "Enable" → Input the applicable 3Shape account.

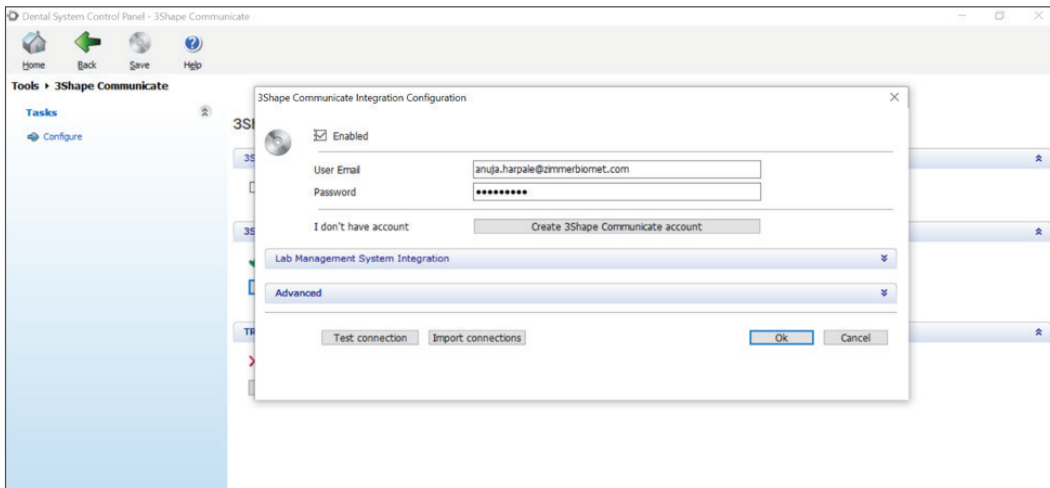

Step 3: Configure 3Shape Dental Manager for Communicate (continued)

4. Adding Connections: Click "Import Connections". Active connections will be imported into Dental Manager as Dentists/Labs/Manufacturers.

The screenshot shows the "3Shape Communicate Integration Configuration" dialog box. It features a "3Shape Communicate" title bar and a close button. The main area includes a "3Shape Communicate Integration Configuration" header, a "CD" icon, and a checked "Enabled" checkbox. Below this are input fields for "User Email" (containing "anuja.harpace@zimmerbiomet.com") and "Password" (masked with dots). A "Create 3Shape Communicate account" button is positioned next to the "I don't have account" text. The dialog has two expandable sections: "Lab Management System Integration" and "Advanced". At the bottom, there are buttons for "Test connection", "Import connections", "Ok", and "Cancel".

5. (Optional) Verify connections: Control Panel → Site Settings → Dentists/Labs/Manufacturers → Find the lab/customer in the left column and click → Under "3Shape Communicate Integration", verify that "Enabled" is selected and the user email has been entered.

The screenshot displays the "3Shape Dental System Control Panel" interface. The "Site Settings" section is active, showing a list of "Labs" on the left and configuration options on the right. The "Labs" list includes "Default Manufacturer", "BIOMET 3I USA", "Michael Dacre Zimmer Biomet ...", "Mike Dacre", "Zimmer Biomet Testing", "ZBD_PBG", "Zimmer Biomet PBG", "Toothoracke Milling Centre", "Zimmer Biomet CS", and "Andrii 3Shape". The "3Shape Communicate Integration" section is expanded, showing "Enabled" checked, "User Email" set to "mikedacre@hotmail.com", and buttons for "Set user Email", "Create Account", and "Connect Site". Other sections like "TRIOS Direct Connection", "ID settings", and "FTP Inbox settings" are collapsed. The interface includes navigation buttons like "Home", "Back", "Save", and "Help" at the top.



For any questions regarding 3Shape Communicate setup,
please contact ZimVie Customer Service by phone at 1-888-800-8045
or by email at DentalCSDigital@ZimVie.com

